

VERTICAL STANDER TRAY, VS1-T

(effective 10/15/02)

These are the user instructions for the VS1-T tray. All persons who set-up, adjust, or use the unit should first read these entire instructions. Please store these instructions for future reference.

USE

The VS1-T is designed as an optional tray for the VS1 and VS1+C Vertical Stander.

VERTICAL STANDER PREPARATION

Refer to the instructions that accompany the vertical stander. Adjust the vertical extension so that there are two or more open holes above the top cushion attachment bolts.

TRAY INSTALLATION

On the underside of the tray there is an aluminum socket centered near one edge. That socket fits over the vertical extension that projects from the stander.

The black knob on the underside of the tray tightens to eliminate side-to-side wobble.

To prepare the tray for installation:

1. If there is a pin projecting into the socket, turn the black knob until the pin no longer projects into the socket.
2. Place the tray on the stander so that the socket fits over the projecting vertical extension. The extension, while inside the socket, should rest against the underside of the tray.
3. Tighten the knob under the tray to prevent side-to-side wobble.

MAINTENANCE AND CARE

Inspect the unit regularly. Tighten the hardware as necessary.

If the unit needs service or spare parts, contact Kaye Products, Inc. or the distributor from whom you purchased the unit.

If a problem is discovered that may impact the unit's function, immediately cease use and contact Kaye Products, Inc.

Do not expose the unit to rain or submerge it in water.

Use a slightly damp cloth to clean the unit, but wipe the dampness from the wood after cleaning it.

Avoid any undue stress to the unit while using, storing, or transporting it.

LIMITED WARRANTY

If an item proves defective within two years of the original purchase, we will provide replacement parts in order to correct that defect. Normal wear and tear is not covered by the warranty.

Kaye Products, Inc. makes no other warranty, expressed or implied, and does not warrant the product as being fit for a particular purpose. The purchaser, owner, and user assume all risk of personal and property injury due to the use of the equipment.

CAUTIONS

1. Do not adjust the unit while it is in use.
2. Before use, ensure that all hardware is securely tightened.
3. Do not use the unit if there are broken or missing parts.
4. Do not alter the unit or use it in any way other than described in these instructions.
5. Do not leave the user unattended.
6. Use qualified supervision.
7. Observe all of the cautions and size and weight limits for the Kaye Vertical Stander.

QUESTIONS?

If you have any questions, comments, or requests, please contact Kaye Products, Inc.



Our authorized representative for the European Medical Device Directives is Advena, Ltd., PO Box 30, Leominster, Herefordshire, HR6 0ZQ, UK.